

The BIOMEASURE II machine has a full dollar bill vault.

The machine does not need to be turned off to empty the vault. Use the key to open the lower door where the vault is located.

On the bottom side of the dollar bill acceptor, there is a red button.

Reach underneath the unit and slide the red button toward the front of the machine.

Slide the vault down about one half of an inch or so, while pressing the red button.

Slide the vault straight back to remove the unit. Use the side flip door to access the money. Remove the cash. Now prepare to re-install the dollar bill vault.

To replace the box, place the plastic pegs back onto the slots push in and slide up. It will click into place once the vault is seated properly.

