

*This is a diagnostic test, confirming that the printer only is functioning. It works on most of the BIOMEASURE printers.*

**PAPER FEED TEST instruction sheet**

Locate the paper feed button on the printer controller board. It is located in between J1 and C10 and is labeled SW2 on the circuit breadboard.

Once you have located the paper feed button, turn the machine off.

Turn the machine back on, *while holding down* the paper feed button.

This will begin the diagnostic test for the printer.

Did the printer print a series of script and characters?

Yes.

No.

The printer portion of the machine is working properly. Move on to the next issue or, conclude the call.

*Please refer to a technician. The printer does not pass the diagnostic test. It may be necessary to replace the printer.*